

# Hyfrecator® Product Catalog











To learn more about these innovative products, visit www.CONMED.com

# **Contents**



Ordering Information	2
Hyfrecator® 2000	3, 4
Hyfrecator <sup>®</sup>	5
Disposable Hyfrecator® Accessories	7
Reusable Hyfrecator® Accessories	8 - 10
Bipolar Accessories.	11, 12
Numerical Index	14
Additional Resources	16 - 18

# **Ordering Information**



#### **To Place Your Order**

Please provide a purchase order number, billing and shipping instructions, product catalog number, description, and pricing to:

#### **Customer Service (Within U.S.)**

Telephone: 866-4-CONMED

(315) 797-8375 ext. 5111

Fax: (800) 438-3051

E-mail: customerservice@conmed.com

#### **ConMed International Customer Service**

Telephone: +1 (315) 797-8375

Fax: +1 (315) 735-6235

E-mail: customerservice@conmed.com

Purchase orders should be addressed to CONMED Corporation, 525 French Road, Utica, NY 13502

#### **Terms and Conditions**

ISO13485

All prices and specifications are subject to change without notice. All prices are FOB Utica, New York, Greenwood Village, Colorado; Lithia Springs, Georgia. (F.O.B. Drop Shipment Points where applicable).

Credit terms: Net 30 days where approved

#### **Return Goods Policy**

All CONMED products, except products that are obsolete, discontinued, or damaged and specialty products, may be returned within 60 days after date of purchase for credit subject to the following:

- 1. A copy of the invoice number and date of purchase and a Return Authorization Number must accompany the returned goods with the reason for the return stated in writing. Contact CONMED Customer Service for the Return Authorization Number at 1-800-448-6506.
- 2. All returned goods would be subject to a minimum of 15% restocking charge. The exact amount of credit to be given, if any, will be determined by a Quality Assurance inspection. Sterile packaged products may be returned for a credit only if returned in the original, unopened shelf package.
- 3. If the return is due to our error in shipping, full credit will be given.
- 4. All returns should be addressed as indicated by Return Authorization.

#### **Quality Policy**

"CONMED is committed to maintaining a quality system that provides for safe and effective products and services that meet the needs and requirements of our patients, customers and company stakeholders."



# Hyfrecator® 2000

#### Hyfrecator® 2000 & Accessories

#### What makes the Hyfrecator® 2000 different?

- Precisely Controlled Power Output Improves clinical outcomes
- Dual Microprocessing Technology Continuously monitors the system for reliability and patient safety

#### Hyfrecator® 2000 ESU (International Units Available)

	Hei. #
Hyfrecator® 2000 115v, Includes power cord	7-900-115
Hyfrecator® 2000 100v, Includes power cord	7-900-100
Hyfrecator® 2000 220v, Power cord sold separately	7-900-220
Hyfrecator® 2000 230-240v, Power cord sold separately	7-900-230

IEC 320 Power Cord Set, European	60-5108-001
IEC 320 Power Cord Set, British	60-5108-002
IEC 320 Power Cord Set, Italian	60-5108-003
IEC 320 Power Cord Set, US	62-5773-001

Box Contents included with the purchase of 115V Hyfecator (#7-900-115):

- (1) Hand-switching handpiece (#7-900-5)
- (1) Wall mounting kit (#7-796-20)
- (1) Power Cord (#62-5773-001)
- (5) Sharp, non-sterile disposable electrodes (7-100-12)
- (5) Blunt, non-sterile disposable electrodes (7-101-12)
- (1) Inservice Training CD (#7-900-25-CD)
- (1) Operators Manual (7-900-OM-ENG)

\*\*International voltages (7-900-100, 7-900-220, 7-900-230) are shipped separately from the power cord and Operator's Manual to accommodate for local outlets and languages.





Dof #







# Hyfrecator® 2000











#### **Training Tools**

Ref. #

Video — "Hyfrecator" 2000 Inservice"

- Provides setup instructions and other helpful tips for using the Hyfrecator®

Video — "Common Dermatologic Procedures"

 Step-by-step instructions are provided by a noted physician for the removal of spider veins, moles, warts, skin tags, seborrheic keratoses, and skin cancers. (Approximate running time is 15 minutes).

Hyfrecator® 2000 Operator's Manual

- Available in English, Spanish, French, Chinese, German, Dutch, Japanese, Swedish, Italian, Portuguese.
- Includes safety precautions, terminology for systems and procedural configurations, applications, maintenance, optional accessories, specifications and performance graphs.

English	
Spanish	7-900-OM-SPN
French	7-900-0M-FRE
Chinese	7-900-OM-CHN
German	7-900-0M-GER
	7-900-OM-DUT
Japanese	7-900-0M-JPN
Swedish	
Italian	7-900-0M-ITN
Portuguese	

- Available in English.
- Guides the qualified biomedical electronics technician through all aspects of preventative maintenance and repairs necessary to keep the Hyfrecator<sup>®</sup> up to original factory specifications.

#### **Mobile/Mounting Accessories**

- Gray, five-legged pedestal stand that can be adjusted from 2-4ft.
- Wall mount brackets are pre-installed.



# **Hyfrecator® Accessories**

#### Pencils/Handpieces

Ref. #



#### **Foot Switch**

#### **Patient Plate**







# Disposable Hyfrecator® Accessories

#### Disposable Hyfrecator® Accessories

#### **Disposable Pencil Sheaths**

Ref. #

Pencil Sheath — Non-sterile Disposable, clear plastic Sheaths protect the pencil body from external conta during procedures. 8in. (20.0cm) long.	amination
100/box	7-796-18BX
12 boxes/case	7-796-18CS
Pencil Sheath — Sterile Disposable, clear plastic sheaths protect the pencil body and portion of cable external contamination during procedures. 36in. (91.44cm) long.	e from
25/box	7-796-19BX
4 boxes/case	7-796-19CS




#### Electrolase Disposable Electrodes — Sharp Tips

Electrolase® Sharp Tips — Non-sterile
Disposable Hyfrecator® tips ideal for pinpoint coagulation.
White plastic base.

100/box	. 7-100-12BX
12 boxes/case	7-100-12CS
Electrolase® Sharp Tips — Sterile Disposable Hyfrecator® tips ideal for pinpoint coagulation. White plastic bas	e.
50/box	7-100-8BX
12 boxes/case	7-100-8CS




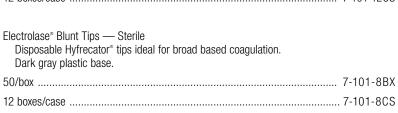



# Reusable Hyfrecator® Accessories



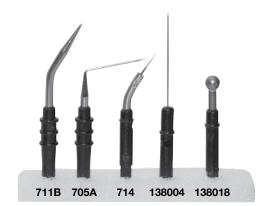


#### Reusable Hyfrecator® Accessories





# Reusable Hyfrecator® Accessories

### **Reusable Needle Electrodes** Ref. # Extra Fine Needle Electrode 30° angled, .375in., 1.0cm, extra fine needle electrode designed for delicate procedures. Commonly used for epilation and telangiectasias dermatology procedures as well as ophthalmology applications. Epilation/Telangiectasias Needle With special coating to reduce build up, this 30° angled, .375in., 1.0cm, extra fine needle electrode designed for delicate procedures. Commonly used for epilation and telangiectasias dermatology procedures as well as ophthalmology applications. Needle Electrode for Pinpoint Procedures 30° angled, .75in. (1.905cm) needle electrode for pinpoint coagulation 2.5in (6.45cm) total length. Long Desiccation and Coagulation Needle .625in. (1.588cm), electrode on long shaft, 5in. (12.7cm) total length. Long Desiccation Needle .375in. (1cm) needle electrode on long shaft, 5in. (13cm) total length. Short Desiccation Needle .375in. (1cm) needle electrode on short shaft, 2in. (5cm) total length. Fine Wire Needle 75° angled, .5in. (1.3cm) needle electrode. Needle Electrode .625in., 2in. total length.

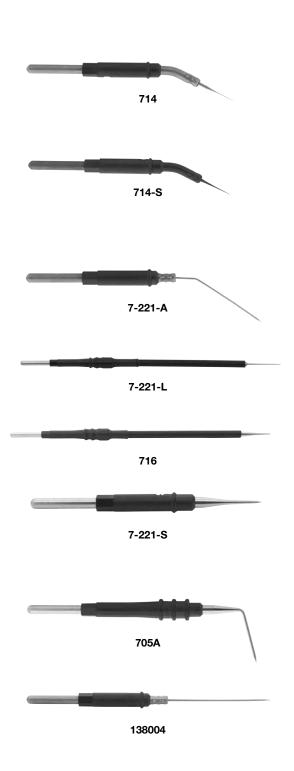

# Reusable Hyfrecator® Accessories











#### **Reusable Ball Electrodes**

5in. (13cm) total length.

Ref. #
Short Ball Electrode
5/pk
0.125in. (.3175cm) diameter ball electrode on short shaft.
Ball Electrode
10/pk
.1875in. (.5cm) diameter ball electrode used for controlled coagulation.
1.6in. total length. Sold in package of 10 each.
Angled Ball Electrode
1/pk
30° angled, .09375in. (.26cm) diameter ball electrode for controlled
coagulation. 2.1in. (5.3cm) total length.
Long Ball Electrode
1/pk
.21875in. (.56cm) diameter ball electrode on long shaft,



# **Bipolar Accessories**

#### **Bipolar Forceps**

For precise desiccation and coagulation control. All forceps are autoclavable and require a bipolar cord and foot switch for operation. Cold sterilization may damage insulating material.

All forceps are sold 1 per pack and requires foot switch and bipolar cord.

Jeweler's Micro Tip, Straight
Semkin Micro Tips, with Stops
Semkin Straight
Gerald Micro Tips
Hardy Micro Tips
Adson Smooth Tips
Cushing Smooth Tips
Cushing Smooth Tips



# **Bipolar Accessories**







Ref. No.



#### **Bipolar Cord**



#### **Foot Switches**



# HYFRECATOR® PRODUCTS INDEX

# **Numerical Index**



Ref. #	Description	Page #
700	Electrode Starter Kit	8
705A	Fine Wire Needle	9
711B	General Purpose Electrode with Protective Insulator	8
714	Extra Fine Needle Electrode	9
714-S	Epilation/Telangiectasias Needle	9
716	Long Desiccation Needle	9
138004	Needle Electrode	9
138018	Ball Electrode	10
138019	Short Ball Electrode	10
60-1771-001	Semkin Micro Tips, with Stops, Bipolar Forcep	11
60-1772-001	Semkin Straight, Bipolar Forcep	11
60-5108-001	IEC 320 Power Cord Set, European	3
60-5108-002	IEC 320 Power Cord Set, British	3
60-5108-003	IEC 320 Power Cord Set, Italian	3
62-5773-001	IEC 320 Power Cord Set, US	3
7-100-12BX	Electrolase* Sharp Tips — Non-sterile, 100/box	7
7-100-12CS	Electrolase Sharp Tips — Non-sterile, 12 boxes/case	7
7-100-8BX	Electrolase Sharp Tips — Sterile, 50/box	7
7-100-8CS	Electrolase Sharp Tips — Sterile, 12 boxes/case	7
7-101-12BX	Electrolase Blunt Tips — Non-Sterile, 100/box	7
7-101-12CS	Electrolase Blunt Tips — Non-Sterile, 12 boxes/case	7
7-101-8BX	Electrolase Blunt Tips — Sterile, 50/box	7
7-101-8CS	Electrolase Blunt Tips — Sterile, 12 boxes/case	7
7-221-A	Needle Electrode for Pinpoint Procedures	9
7-221-L	Long Desiccation and Coagulation Needle	9
7-221-S	Short Desiccation Needle	9
7-222-A	Angled Ball Electrode	10
7-222-L	Long Ball Electrode	10
7-796-18BX	Disposable Pencil Sheaths, 100/box	7
7-796-18CS	Disposable Pencil Sheaths, 12 boxes/case	7
7-796-19BX	Pencil Sheath — Sterile , 25/box	7
7-796-19CS	Pencil Sheath — Sterile, 4 boxes/case	7
7-796-20	Wall Mount Kit	4
7-809-1	Jeweler's Micro Tip, Straight, Bipolar Forcep	11
7-809-12	Bipolar 12ft. (3.7cm) Cord	12
7-809-2	Gerald Micro Tips, Bipolar Forcep	11
7-809-3	Hardy Micro Tips, Bipolar Forcep	11
7-809-4	Adson Smooth Tips, Bipolar Forcep	11
7-809-6	Cushing Smooth Tips, Bipolar Forcep	11
7-809-7	Adson Serrated Tips, Bipolar Forcep	12
7-809-8	Cushing Serrated Tips, Bipolar Forcep	12
7-809-9	Cushing Serrated Tips, Bipolar Forcep	12
7-900-1	Telescoping Mobile Hyfrecator® Stand	4



# **Numerical Index**

Ref. #	Description	Page #
7-900-100	Hyfrecator® 2000 100v, Includes power cord	3
7-900-115	Hyfrecator® 2000 115v, Includes power cord	3
7-900-220	Hyfrecator® 2000 220v, Power cord sold separately.	3
7-900-230	Hyfrecator® 2000 230-240v, Power cord sold separately	3
7-900-4	Foot Switch	5, 12
7-900-5	Autoclavable, Reusable Hand Control Pencil	5
7-900-6	Autoclavable, Reusable Foot Control Pencil	5
7-900-7	Patient Plate	5
7-900-0M-CHN	Hyfrecator® 2000 Operator's Manual Chinese	4
7-900-0M-DUT	Hyfrecator® 2000 Operator's Manual Dutch	4
7-900-0M-ENG	Hyfrecator® 2000 Operator's Manual English	4
7-900-0M-FRE	Hyfrecator® 2000 Operator's Manual French	4
7-900-0M-GER	Hyfrecator® 2000 Operator's Manual German	4
7-900-0M-ITN	Hyfrecator® 2000 Operator's Manual Italian	4
7-900-0M-JPN	Hyfrecator® 2000 Operator's Manual Japanese	4
7-900-0M-POR	Hyfrecator® 2000 Operator's Manual Portuguese	4
7-900-0M-SPN	Hyfrecator® 2000 Operator's Manual Spanish	4
7-900-0M-SWE	Hyfrecator® 2000 Operator's Manual Swedish	4
7-900-SM-ENG	Hyfrecator® 2000 Service Manual	4

## **Additional Resources**



#### **Applications for Dermatology**

#### **Benign Lesions**

The Hyfrecator® 2000 can destroy a variety of benign skin lesions. With its wide range of power settings, the Hyfrecator® 2000 treats even the most delicate facial lesions as well as the thickest scaling lesions of the trunk. Some of the clinical applications include:

- Acrochordon (Skin Tag)
- Actinic Keratoses
- Adenoma Sebaceum
- Angiokeratoma
- Angiomas, Capillary
- Angiomas, Spider

- Condyloma Acuminatum (Veneral Wart)
- Fibroma
- Keratoacanthoma
- Lymphangioma
- Molluscum Contagiosum
- Pyogenic Granuloma

- Seborrheic Keratoses
- Sebaceous Papules Syringomas
- Telangiectasias
- Common Warts (Verrucae Vulgaris)
- Filiform Warts
- Flat Warts (Verrucae Plana)

#### Malignant Lesions

If malignancy is suspected, a biopsy is recommended before treatment by Electrosurgery for histopathologic examination. The most common skin cancers are basal and squamous cell carcinomas. Appropriately selected ones may be treated easily, quickly and effectively with curettage and electrofulguration. When lesions are chosen appropriately, very acceptable cure rates for Electrosurgery can be achieved. In addition, Electrosurgery has the advantage of being easy to learn, simple to perform and cost-effective. Cosmeses is quite acceptable and in many cases, may be preferable to the cosmetic results of excisional surgery or radiotherapy.

- Basal Cell Carcinoma
- Bowen's Disease (Squamous Cell Carcinoma in Situ)
- Bowenoid Papulosis
- Squamous Cell Carcinoma

#### Other Applications

#### **Plastic Surgery**

Many surgeons use the Hyfrecator\* 2000 for hemostasis in plastic and reconstructive surgery. It saves time and produces minimal tissue reaction compared to other methods. Coagulation occurs by touching each bleeding point with an electrodessicating current. Because blood will dissipate the energy, a sponge should be used just prior to application.

#### Gynecology

- Vaginal Cysts
- Condylomata of the Vulva
- Cervical Polyps
- Pruritus Vulvae
- Urethral Tumors
- Cervical Erosion
- Cervicitis
- Bartholin's Cysts or Abscesses
- Nabothian Cyst
- Skene's Glands

#### Ophthalmology

- Chalazion (Meibomian Cysts)
- Entropion
- Xanthelasma
- Burn Entropion
- Spastic Entropion

#### Otolaryngology

- Adenoid-Hypertrophied Remnants
- Epistaxis
- Granular Pharyngitis
- Nasal Polyps
- Hemostasis in Tonsilloadenoidectomy
- Tonsil Tag Destruction
- Turbinate Shrinkage

#### **Proctology**

- Fissure-in-ano
- Hemmorrhoids
- Ischiorectal
- Papilloma

#### Urology

- Bladder Tumor Ablations
- Vasectomy
- Veneral Warts

#### **Dental and Oral Surgery**

- Apioectomy
- Buccal Gingival Cavity
- Cysts, Mucous
- Dentin Desensitization
- Frenectomy
- Hemostasis
- Gingivectomy
- Root Canal Sterilization

<sup>\*</sup>The above represents a sample of procedural applications developed by Hyfrecator\* users. It is recommended that users attend training courses, as appropriate, to develop proper surgical techniques.



## **Additional Resources**

#### **Glossary of Terms**

The following glossary of terms has been provided in order to familiarize yourself with the terminology used in the video training tape.

#### Coagulation

The desiccation of tissue via high-frequency generated electrical current.

#### **Desiccation**

A common form of superficial electrosurgery where the treatment electrode LIGHTLY CONTACTS the tissue surface. Derived from the Latin term desiccare, which means "to dry".

#### **Fulguration**

Superficial Electrosurgery in which the treatment electrode is not in contact with the tissue, and a spark of high-frequency electrical current jumps between the electrode and the tissue. Derived from the Latin term fulgur, which means "lightning".

#### **Electrode**

In electrosurgery, an instrument with a point or surface from which to transmit an electrical current, i.e. disposable electrodes, reusable electrodes.

#### **Bipolar**

Electrodes having two poles with electric current alternating between both poles, i.e. forceps.

#### **Epilation**

The removal of hair. One form of epilation utilizes the Hyfrecator and is termed "thermolysis" (Utilize the 714 epilation needle set).

#### Seborrheic Keratoses

Perhaps the most common skin tumors in adults, often associated with old age.

#### Telangiectasia

A condition consisting of superficial, visible blood vessels. Although often referred to as broken blood vessels or spider veins, they are normal capillaries. Common areas are the face, nose, side of the neck and upper chest and legs.

#### Nevi (Mole)

This term is used to designate a pigmented fleshy growth and is applied loosely to any blemish of the skin.

For further information and product support, visit www.hyfrecator.com







# CONMED Hyfrecator® Product Catalog

11311 Concept Blvd. Largo, FL 33773

Toll Free: 1-866-4CONMED International: 727-214-3000 www.CONMED.com

customerexperience@CONMED.com internationalorders@CONMED.com