



## Reflex® Skin Staplers

- Two stapler styles: Reflex® One and Reflex® TL
- Two staple sizes: regular and wide
- Confirmation of complete staple formation
- Enhanced visibility
- Visible staple counter for more efficient use



Reflex® One



Reflex® TL



Reflex® Extractor



Rotating Head Stapler

**AIRSEAL® iFS**  
INSUFFLATION  
MANAGEMENT SYSTEM



**HELIXAR™ ESU**  
WITH ARGON BEAM  
COAGULATION



**HANDHELD  
INSTRUMENTS**  
LESS INVASIVE APPROACH



To learn more about these and other innovative products, call **866-4CONMED** or visit **www.CONMED.com**

# Reflex<sup>®</sup> Skin Staplers

Removable skin staples are used to close external skin under high tension and are designed to hold skin in place until sufficient wound healing permits staple removal.

CONMED offers three lines of disposable, user-friendly Reflex<sup>®</sup> Skin Staplers that deliver a superior external skin closure performance as well as an extractor for sufficient staple removal. Reflex<sup>®</sup> Skin Staplers are available in regular or wide staple sizes, supporting a variety of skin closures.

## Overview and Features



Reflex <sup>®</sup> One	Reflex <sup>®</sup> TL (Tissue Lift)	Reflex <sup>®</sup> RHS	Reflex <sup>®</sup> Extractor
<ul style="list-style-type: none"> <li>Fully retracting anvil</li> <li>Audible and tactile reinforcement</li> <li>35 staples included</li> <li>Visible staple counter</li> <li>Single patient use</li> <li>Regular or Wide</li> </ul>	<ul style="list-style-type: none"> <li>Semi-retracting anvil</li> <li>Audible and tactile reinforcement</li> <li>35 staples included</li> <li>Visible staple counter</li> <li>Single patient use</li> <li>Regular or Wide</li> </ul>	<ul style="list-style-type: none"> <li>360° rotating head</li> <li>Audible and tactile reinforcement</li> <li>35 staples included</li> <li>Visible staple counter</li> <li>Single patient use</li> <li>Wide</li> <li>Reloadable</li> </ul>	<ul style="list-style-type: none"> <li>Ergonomic fit</li> <li>Supports staple extraction</li> <li>Compatible with wide and regular staples</li> </ul>



### Trigger

- Ergonomic
- Easy to use
- Audible and tactile reinforcement

### Anvil

- Holds staple in place until trigger is released
- Helps form staple shape

### Staple Counter

- Easily identify how many staples remain
- Clear visual guide

### Arrow Guide


- Assists with placement of staples

### Nose Piece



- Open view and low profile for improved staple visibility

# Ergonomics

CONMED's Reflex® One and Reflex® TL ergonomic design provides more area between the trigger and the skin. This keeps the nose of the stapler parallel to the skin allowing proper staple formation and tissue approximation.

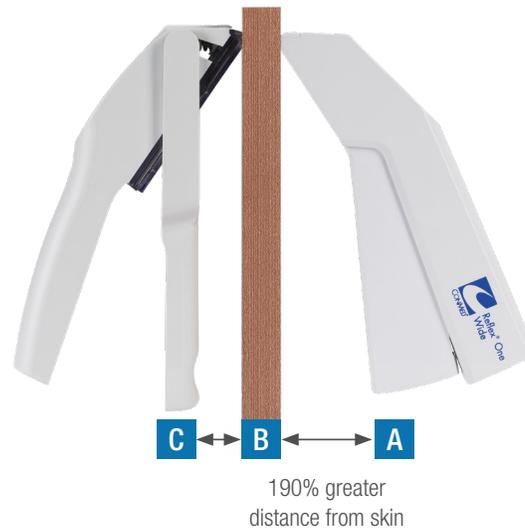

CONMED Reflex® versus "Other" Leading Brand  
**A to B Distance:** 1.947" **C to B Distance:** .672"

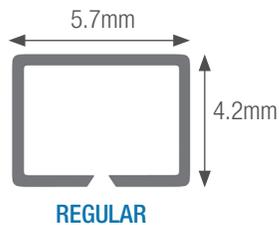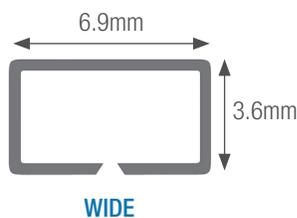

**Staples are formed from 316L surgical-grade stainless steel.**

When it is not possible to maintain at least 5mm distance from the stapled skin to underlying bones, vessels, and internal organs, the use of staples for skin closure is contraindicated.

## **MRI Safety Information**

The Reflex® One and Reflex® TL Skin Staplers have not been evaluated for safety in the MR environment. It has not been tested for heating or unwanted movement in the MR environment. The safety of the Reflex One and Reflex TL Skin Staplers in the MR environment is unknown. Performing an MR exam on a person who has this medical device may result in injury or device malfunction.

## Ordering Information

Description	Catalog Number
<b>Reflex® One Skin Stapler</b> 35 Wide, 6/box	8535
<b>Reflex® One Skin Stapler</b> 35 Regular, 6/box	8035
<b>Reflex® TL Skin Stapler with Tissue Lift</b> 35 Wide, 6/box	8735
<b>Reflex® TL Skin Stapler with Tissue Lift</b> 35 Regular, 6/box	8635
<b>Reflex® Extractor</b> Wide / Regular, 12/box	3031
<b>Reflex® Rotating Head Skin Stapler</b> 35 Wide, 6/box	8935
<b>INTERNATIONAL ONLY</b>	
<b>Reflex® TL Skin Stapler with Tissue Lift</b> 35 Wide, 6/box	3013
<b>Reflex® TL Skin Stapler with Tissue Lift</b> 35 Regular, 6/box	3013R
<b>Reflex® One Skin Stapler</b> 35 Wide, 6/box	3036
<b>Reflex® One Skin Stapler</b> 35 Regular, 6/box	3036R